

The Injury Free Ampoule Stand



Stop Needle Stick & Sharps Injuries

Users are protected from dangerous injuries while processing glass ampoules.



Prevent Costly & Dangerous Spills

Stable ergonomic design virtually eliminates ampoule compound spills.




Available for Any Ampoule Design

Including all DIN, OPC, fully scored, and custom types. Contact us to discuss your ampoule requirements.



Stop Drug Mix-Up

The inherent design of the AS⁺ keeps ampoules, syringes, and needles organized to minimize user error.

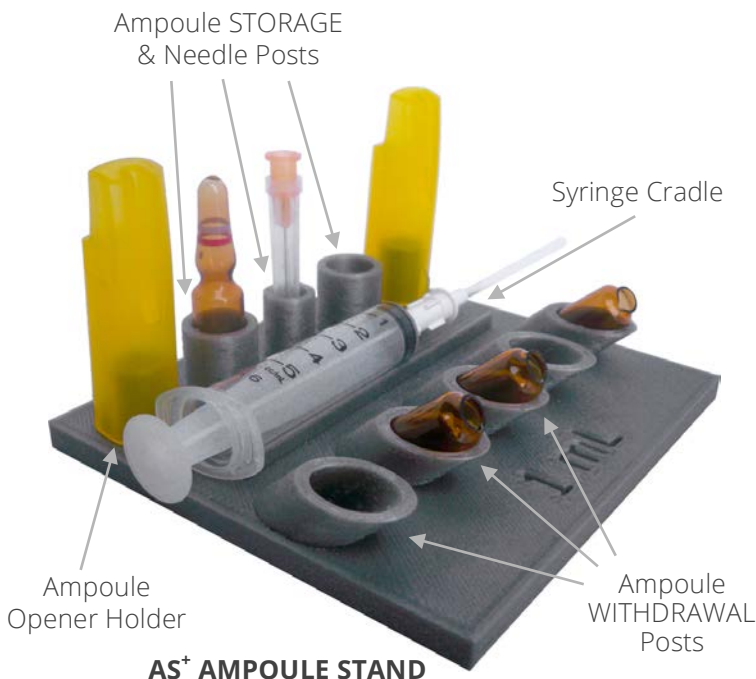

Using the AS⁺

1. Place an opened ampoule into an ergonomically angled Ampoule WITHDRAWAL Post.
2. After attaching a filter needle/straw to a syringe, safely withdrawal the ampoule solution while easily keeping your hands away from danger.
3. Swap the original filter needle/straw with an insertion needle/straw from the safety of the needle posts to avoid needle stick injuries.
4. Remove air bubbles and excess compound from the syringe and place it in the Syringe Cradle.
5. Congratulations! You've safely prepared an ampoule compound for use while virtually eliminating sharps injuries and drug mix-ups.

AS⁺ Versus Other Ampoule Processing Methods

	AS ⁺	Traditional Stands	Bare Hands
Reusable	✓	✓	✓
Minimizes Costly & Dangerous Spills	✓	✓	✗
Stops Incorrect Drug Admin from User Error	✓	✗	✗
Open Ampoules Directly on the Stand*	✓	✗	✗
Ergonomic Angled Design for Easy Use	✓	✗	✗
Works with Any Ampoule**	✓	✗	✗
Industry Leading ROI***	✓	✗	✗
100% Protection from Needle Stick and Sharps	✓	✗	✗

* Compatible with the AO⁺.

** Individual customization of the AS⁺ may be required.

*** ROI is achieved by lowering costs associated with fatigue, injuries, and errors.



STEP 1: PREPARE



STEP 2: WITHDRAWAL



STEP 3: CHANGE NEEDLES



TEMPERATURE WARNING

Structural damage to the AO⁺ will occur at temperatures above 70° C (160° F). Please contact sales@theampouleopener.com for technical questions.

For information or if you wish to offer a testimonial, please contact: sales@theampouleopener.com.

USA Direct Telephone: 802-735-2557

Global Sales Manager: John Bethune
(john@theampouleopener.com)



A Word of Caution: There is inherent risk in opening glass ampoules. AS⁺ is designed to minimize risk, yet variances among manufacturers, supply quality, and dynamic forces inherent in opening glass ampoules and using syringes can produce unpredictable results. Always exercise caution.